


OTOVENT® MIDDLE EAR AUTO INFLATION SYSTEM

The proven alternative to the surgical treatment of symptoms of Otitis Media and Eustachian Tube Dysfunction.

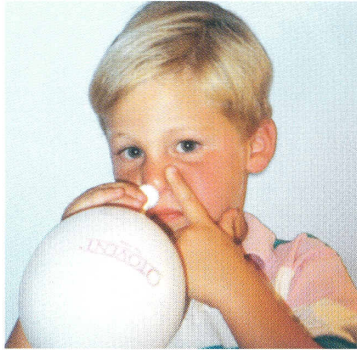

Used around the world, for more than 18 years, the Otovent Middle Ear Auto Inflation System offers Otitis Media Patients and those suffering from Barotrauma a proven protocol to equalize middle ear pressure and relieve painful symptoms. The Valsalva and Politzer treatments have long been used to equalize middle ear pressure in the presence of negative pressure pain of Otitis Media, Barotrauma due to underwater pressure, air flight, or hyperbaric chamber treatments.

BENEFITS OF OTOVENT

Otovent can prevent or treat Otitis Media and be an attractive alternative to surgery. Otovent treats many of the complications associated with negative ear pressure caused by Eustachian tube dysfunction.

- Frequent treatments are painless and regarded as "fun" by children
- Typically satisfying results early on; compliance is very high
- Effective without drugs and with no known side effects
- May avoid or delay recourse to surgery or ventilation tubes
- Very low treatment cost


BENEFITS OF OTOVENT

Otovent® is simple to use. With one nostril pinched shut, and the balloon mounted nose plug held against the other nostrum, the patient inflates the balloon. Once fully extended, the patient then allows the positive pressure of the balloon to reenter the nose, and swallows, achieving trans-Eustachian tube middle ear insufflation.

BEFORE: Otitis Media, Eustachian tube dysfunction
Negative pressure, pain, impaired hearing, learning problems.

DURING: Controlled pressure is transmitted through the nose and Eustachian tube to the middle ear pressure is balanced.

AFTER: Pain relief, hearing restored and improved prognosis.

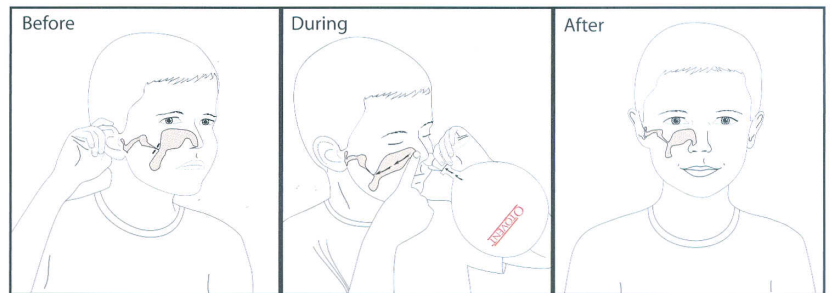

OTOVENT TREATMENT SET

The system consists of a nares bulb and an ample supply of latex balloons to perform safe and effective middle ear auto inflation, 2-3 times daily, for up to 6 weeks.

Each Kit Contains: one nose piece made of PVC, one instruction leaflet, and 10 pressure tested latex balloons



Number	Description	Qty/Box
90-00000	Otovent Kits, 1 Case*	12 patient kits
90-00001	Otovent Kits, 1/2 Case*	6 patient kits
90-00000B	Otovent balloons*	5 per pack

The system consists of a nares bulb and an ample supply of latex balloons to perform safe and effective middle ear auto inflation, 2-3 times daily, for up to 6 weeks.

* Contains latex